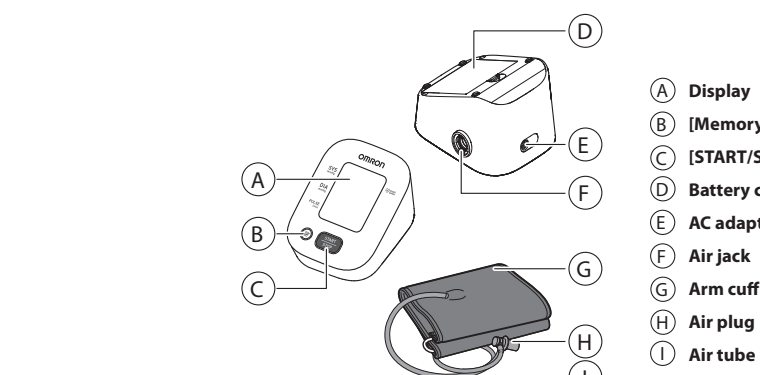
Automatic Upper Arm Blood Pressure Monitor M2+ (HEM-7146-E)

Read Instruction manual 1 and 2 before use.

For symbols information, refer to "Symbols Description" on the back side of the another "Instruction manual 1".

All for Healthcare

Introduction Thank you for purchasing the M2+ Automatic Upper Arm Blood Pressure Monitor. This product prescribes the most accurate method of blood pressure measurement.

Safety Instructions This instruction manual provides you with important information about the M2+ Automatic Upper Arm Blood Pressure Monitor.

Intended Use This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population.

Receiving and Inspection Remove this monitor from the packaging and inspect for damage.

Important Safety Information Read the Important Safety Information in this instruction manual before using this monitor.

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express their pain.
DO NOT adjust medication based on readings from this blood pressure monitor.

General Precautions

When you take a measurement on the right arm, the right arm should be at the side of your elbow.

Battery Handling and Usage

DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged.

Specifications

Table with 3 columns: Display/Problem, Possible Cause, Solution. Includes items like 'Er appears', 'Batteries are low', 'Nothing appears on the display', 'Readings appear too high or too low'.

Automatic Upper Arm Blood Pressure Monitor M2+ (HEM-7146-E)

Read Instruction manual 1 and 2 before use.

For symbols information, refer to "Symbols Description" on the back side of the another "Instruction manual 1".

All for Healthcare

Introduction Thank you for purchasing the M2+ Automatic Upper Arm Blood Pressure Monitor. This product prescribes the most accurate method of blood pressure measurement.

Safety Instructions This instruction manual provides you with important information about the M2+ Automatic Upper Arm Blood Pressure Monitor.

Intended Use This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population.

Receiving and Inspection Remove this monitor from the packaging and inspect for damage.

Important Safety Information Read the Important Safety Information in this instruction manual before using this monitor.

Warning Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express their pain.
DO NOT adjust medication based on readings from this blood pressure monitor.

General Precautions

When you take a measurement on the right arm, the right arm should be at the side of your elbow.

Battery Handling and Usage

DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged.

Specifications

Table with 3 columns: Display/Problem, Possible Cause, Solution. Includes items like 'Er appears', 'Batteries are low', 'Nothing appears on the display', 'Readings appear too high or too low'.

4. Limited Warranty

Thank you for buying an OMRON product. This product is constructed of high quality materials and great care has been taken in its manufacturing.

5. Guidance and Manufacturer's Declaration

This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., Japan.

2. Maintenance

To protect your monitor from damage, follow the directions below:

2.1 Maintenance

Change or modifications not approved by the manufacturer will void the user warranty.

2.2 Storage

Remove the arm cuff from the monitor 3.

3. Specifications

Table with 3 columns: Product category, Product description, Model (Code). Includes details like 'Electronic Sphygmomanometers', 'Automatic Upper Arm Blood Pressure Monitor', 'M2+ (HEM-7146-E)', 'LCD digital display', '0 to 299 mmHg', 'SYS: 60 to 260 mmHg', 'Diastolic blood pressure range: 40 to 180 mmHg', 'Accuracy: Pressure: ±3 mmHg, Display: ±5% of display reading', 'Inflation: Automatic by electric pump', 'Automatic pressure release valve', 'Measurement method: Oscillometric method', 'Operation mode: Continuous operation', 'IP classification: IP20', 'Rating: DCEV 4 W', 'Power source: 4 x AA batteries 1.5 V or optional AC adapter (INPUT AC 100 - 240 V 50 - 60 Hz 0.12 - 0.065 A)', 'Battery life: Approximately 500 measurements (Using new alkaline batteries and included arm cuff. Depending on the type of battery and arm cuff)', 'Durable period (Service life): Monitor: 5 years, Arm cuff: 5 years, Optional AC adapter: 5 years', 'Operating conditions: 10 to 40°C (50 to 104°F) non-condensing / 80 to 106°F (27 to 42°C)', 'Storage / Transport conditions: -20 to +60°C / -10 to 90°F (RH non-condensing)', 'Weight: Monitor: approximately 250 g (8.8 oz) (including arm cuff), Arm cuff: approximately 170 g (6.0 oz) (including arm cuff), Monitor: 103 mm (W) x 82 mm (H) x 140 mm (D), Arm cuff: 145 mm x 594 mm (air tube: 750 mm)', 'Cuff circumference applicable to the monitor: 17 to 42 cm (included arm cuff: 17 to 42 cm)', 'Memory: Stores up to 30 readings', 'Contents: Monitor, arm cuff (HEM-RM31), 4 "AA" batteries, storage case, Instruction Manual (1 and 2), Instruction manual (2)', 'Protection against electric shock: Internally powered ME equipment (when using only batteries)', 'Applied part: Type BF arm cuff', 'Note: These specifications are subject to change without notice. This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2014 and EN ISO 81060-2:2014 A1:2020 (excluding pregnant and pre-eclampsia patients). In the clinical validation study, KS was used on 85 subjects for determination of manual blood pressure. *IP classification is degrees of protection provided by enclosures in accordance with IEC 60529. The monitor and its accessories are protected against solid foreign objects of 1.25 mm diameter and greater such as a finger. The optional AC adapter HEM-CM01 is protected against vertically falling water drops which may cause issues during a normal operation. The optional AC adapter HEM-CP01 is protected against splashing water drops which may cause issues during a normal operation. *Operation mode classification complies with IEC 60601-1.

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